

1 CareKit Package Contents

Open your CareKit package and check that the contents are complete, unopened, and free from damage.

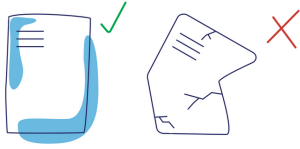The contents will include: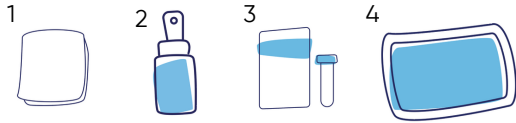

1) gauze sponge(s)
2) wound irrigation solution

3) collagen powder
4) protective composite dressing

2 Get Ready

Take one of each of the components from your CareKit package, and lay them out on a clean surface. Thoroughly wash hands with soap and water.


Note: Take care to ensure your hands are clean before interacting with the wound area in all steps moving forward.

3 Remove Old Dressing

Remove and discard any protective (outer) dressing covering the wound area.

NOTE: Do not forcibly remove any collagen remaining in the wound area.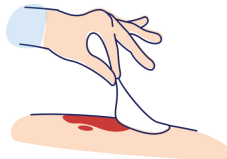

4 Cleanse


Spray the wound area with the wound irrigation solution until it is thoroughly cleansed. No rinse required. You may use a gauze sponge to help wipe up any excess collagen still in the area at this time.


5 Apply New Dressing

Open the pouch of collagen powder and apply directly to the wound area.

NOTE: Pinch the sides of the pouch to facilitate a more precise application on the wound. Sprinkle directly onto the area (~1/4 inch thick to the surface), ensuring it is not overly packed.

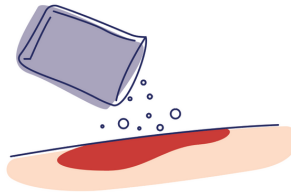
The wound irrigation solution from Step 4 may be sprayed to moisten the collagen before covering the wound in step 6.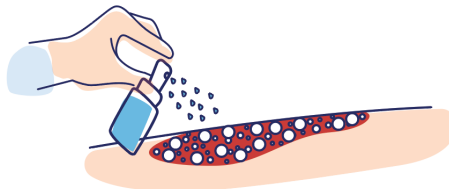


6 Cover & Protect

Use a gauze sponge to gently wipe up the excess collagen / moisture around the wound area.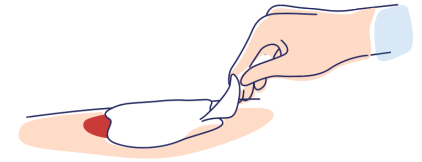

Cover the dressed wound area with the protective dressing, smoothing the edges to secure it in place. The adhesive border should not come in contact with the wound area.

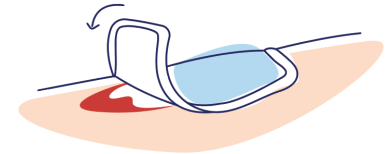NOTE: To ensure a good seal, make sure the area around the wound is dry. If it is damp or wet, gently blot or wipe with gauze.

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